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CE0123 CE Marking of Conformity

**2. Important Safety Instructions**

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

**Never immerse this device in water or other liquids. For cleaning please follow the instructions in the «Cleaning and Disinfecting» section.** Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.

Protect it from:

- water and moisture
- extreme temperatures
- impact and dropping
- contamination and dust
- direct sunlight
- heat and cold

The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m (according to 60601-1-2 table 5). In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.

Do not use the device in an MRI or CT environment. This device is not intended for continuous monitoring.

This device does not have an alarm function and is therefore not suitable for evaluating medical results. Do not use this device in situations where alarms are required. Do not sterilize this device using autoclaving or ethylene oxide sterilizing. This device is not intended for sterilization.

If the device is not going to be used for a prolonged period the batteries should be removed.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes. This device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

**3. General Description**

Oxygen saturation indicates the percentage of hemoglobin in arterial blood that is loaded with oxygen. This is a very important parameter for the respiratory circulation system. Many respiratory diseases can result in lower oxygen saturation within human blood.

**Following factors can reduce oxygen saturation:** Auto-medic regulation of organ dysfunction caused by anaesthesia, intensive postoperative trauma, injuries caused by some medical examinations. These situations may result in light-headedness, asthma and vomiting. Therefore, it is very important to know the oxygen saturation of a patient so that doctors can detect problems in a timely manner.

**4. Measurement Principles**

Principle of this fingertip pulse oximeter: A mathematical formula is established making use of Lambert Beer Law according to spectrum absorption characteristics of deoxygenated hemoglobin (Hb) and oxyhemoglobin (HbO<sub>2</sub>) in red and near-infrared zones.

**Operation principle of this device:** Photoelectric oxyhemoglobin inspection technology is adopted in accordance with capacity pulse scanning and recording technology, so that two beams of different wavelength of lights (660 nm red and 905 nm near infrared light) can be focused onto a human nail tip through a clamping finger-type sensor. A measured signal obtained by a photoelectric element, will be shown on the display through a sensor electronic circuit and microprocessor.

**Diagram of Operation Principle:**

1. Red and infrared-emission tube.
2. Red and infrared-recv receipt tube.

**5. Directions for Use**

1. Insert the batteries as described in the «inserting the batteries (» section).
2. Insert one finger (nail side up, index or middle finger is recommended) into the finger opening of the device. Be sure to fully insert the finger so that the sensors are completely covered by the finger.
3. The device will allow you to clamp down on the finger.
4. Release the ON/OFF button ( ) to turn the device on.
5. Do not shake your finger during the test. It is recommended that you do not move your body whilst taking a reading.

Serial Number / Seriennummer / Sarjanumero / Seriennumeri / Seriennummer / Sérias nomen / Sérias nomers / Серийный номер / Серийный номер

Date of Purchase / Inköpsdatum / Ostapöivämäärä / Kobsdato / Kuppsdato / legokdes datums / Pardavimo data / Ostukupavardė /Дата покупки / Дата покупки

Specialist Dealer / Återförsäljare / Alan kauppias / Special-Forhandler / Specialist forhandler / Specialists -pärstävijä / Pardavusis istaigis / Агентиќ микросгидинга / Спецализированный дилер

Beijing Choice Electronic Technology Co., Ltd.  
2nd Floor, 3rd Floor and Indoor 410-412 414th Floor,  
No. 2 Building, No. 9 Shuangyuan Road,  
Beijingshan, Beijing, 100041 Beijing, People's Republic of China

Shanghai International Holding Corp. GmbH (EUROPE)  
Eiffelstrasse 80, 20537 Hamburg / Germany

**CE0123**

**Pulse Oximeter**

1. ON/OFF button
2. Oxygen saturation (value as percentage)
3. Pulse rate (value in beats per minute)
4. Pulse wave (plethysmographic wave)
5. Pulse bar
6. Low battery indicator
7. Inserting the batteries
8. Attaching the lanyard
9. Display modes
10. Operation principle

Dear Customer,  
This Microlife fingertip pulse oximeter is a portable non-invasive device intended for spot-checking of the oxygen saturation of arterial blood (SpO<sub>2</sub>) and pulse rate of adults and pediatric patients. It is suitable for private use (at home, or on the go) as well as for use in the medical sector (hospitals, hospital-type facilities). It has been clinically proven to be of high precision during repeatability.

If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at [www.microlife.com](http://www.microlife.com) where you will find a wealth of available information on our products. Retain instructions in a safe place for future reference. Stay healthy – Microlife AG!

**1. Explanation of Symbols**

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Read the instructions carefully before using this device.

Type BF applied part

Keep dry

Manufacturer

Date of manufacture

No SpO<sub>2</sub> alarm

Low battery indicator

Serial number

Authorized against dripping water

Authorized representative in the European community

% SpO<sub>2</sub> Oxygen saturation (value as percentage)

PR bpm Pulse rate (value in beats per minute)

Operating conditions:  
5 - 40 °C / 41 - 104 °F

Storage conditions:  
-20 - +55 °C / -4 - +131 °F

**CE0123** CE Marking of Conformity

**2. Important Safety Instructions**

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

**Never immerse this device in water or other liquids. For cleaning please follow the instructions in the «Cleaning and Disinfecting» section.** Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.

Protect it from:

- water and moisture
- extreme temperatures
- impact and dropping
- contamination and dust
- direct sunlight
- heat and cold

The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m (according to 60601-1-2 table 5). In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.

Do not use the device in an MRI or CT environment. This device is not intended for continuous monitoring.

This device does not have an alarm function and is therefore not suitable for evaluating medical results. Do not use this device in situations where alarms are required. Do not sterilize this device using autoclaving or ethylene oxide sterilizing. This device is not intended for sterilization.

If the device is not going to be used for a prolonged period the batteries should be removed.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes. This device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

**3. General Description**

Oxygen saturation indicates the percentage of hemoglobin in arterial blood that is loaded with oxygen. This is a very important parameter for the respiratory circulation system. Many respiratory diseases can result in lower oxygen saturation within human blood.

**Following factors can reduce oxygen saturation:** Auto-medic regulation of organ dysfunction caused by anaesthesia, intensive postoperative trauma, injuries caused by some medical examinations. These situations may result in light-headedness, asthma and vomiting. Therefore, it is very important to know the oxygen saturation of a patient so that doctors can detect problems in a timely manner.

**4. Measurement Principles**

Principle of this fingertip pulse oximeter: A mathematical formula is established making use of Lambert Beer Law according to spectrum absorption characteristics of deoxygenated hemoglobin (Hb) and oxyhemoglobin (HbO<sub>2</sub>) in red and near-infrared zones.

**Operation principle of this device:** Photoelectric oxyhemoglobin inspection technology is adopted in accordance with capacity pulse scanning and recording technology, so that two beams of different wavelength of lights (660 nm red and 905 nm near infrared light) can be focused onto a human nail tip through a clamping finger-type sensor. A measured signal obtained by a photoelectric element, will be shown on the display through a sensor electronic circuit and microprocessor.

**Diagram of Operation Principle:**

1. Red and infrared-emission tube.
2. Red and infrared-recv receipt tube.

**5. Directions for Use**

1. Insert the batteries as described in the «inserting the batteries (» section).
2. Insert one finger (nail side up, index or middle finger is recommended) into the finger opening of the device. Be sure to fully insert the finger so that the sensors are completely covered by the finger.
3. The device will allow you to clamp down on the finger.
4. Release the ON/OFF button ( ) to turn the device on.
5. Do not shake your finger during the test. It is recommended that you do not move your body whilst taking a reading.

Serial Number / Seriennummer / Sarjanumero / Seriennumeri / Seriennummer / Sérias nomen / Sérias nomers / Серийный номер / Серийный номер

Date of Purchase / Inköpsdatum / Ostapöivämäärä / Kobsdato / Kuppsdato / legokdes datums / Pardavimo data / Ostukupavardė /Дата покупки / Дата покупки

Specialist Dealer / Återförsäljare / Alan kauppias / Special-Forhandler / Specialist forhandler / Specialists -pärstävijä / Pardavusis istaigis / Агентиќ микросгидинга / Спецализированный дилер

Beijing Choice Electronic Technology Co., Ltd.  
2nd Floor, 3rd Floor and Indoor 410-412 414th Floor,  
No. 2 Building, No. 9 Shuangyuan Road,  
Beijingshan, Beijing, 100041 Beijing, People's Republic of China

Shanghai International Holding Corp. GmbH (EUROPE)  
Eiffelstrasse 80, 20537 Hamburg / Germany

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